

Cysto-Gastro-Sets

ENDO-FLEX Cysto-Gastro-Sets/Cystotomes are designed to puncture a hole in the transgastric or transduodenal wall and into a pancreatic pseudocyst by involving endoscopic ultrasound (EUS) guidance and fluoroscopy.

Fr. 10 and Fr. 8 type

After the initial access with the implemented needle knife, the incision can be enlarged with the diathermic metal tip on the Fr. 10 or Fr. 8.5 outer sheath. The sets are equipped with two HF connectors, one each for the needle knife and the diathermic tip on the outer catheter.

Fr. 6 type

Specifically designed for smaller working channels, the Fr. 6 Cystotome Has a diathermic metal tip on the Fr. 6 catheter and one HF connector only. It is advanced over a .035" guide wire to the transgastric or transduodenal wall directly.

Drainage

After access to the identified pseudocyst appropriate stents can be placed over the guide wire to facilitate drainage from the cyst to either the stomach or the duodenum.

134 °C
273 °F

Cysto-Gastro-Set

- Cysto-Gastro-Set consisting of needle knife and outer catheter with diathermic tip
- 2 HF-connectors
- Fr. 8.5 Set for guide wire .025 inch
- Fr. 10 Set for guide wire .035 inch



length

Fr. 8.5

Fr. 10.0

180 cm

XS99900901341

XS99900901340



Cysto-Gastro-Set / Cystotome

- Cystotome FR. 6 with diathermic tip
- for guide wire .035 inch
- 1 HF-connector



length

Fr. 6.0

180 cm

XS99900901345



reusable

Cysto-Gastro-Set